

# QUICKSTART GUIDE


## PEROXigard® Concentrate



Peroxigard® Concentrate can be diluted at specific ratios based on the type of application you require. Excellent for daily use and outbreak situations.\* For more information on a specific organism, visit the Additional Claims section on the Peroxigard Concentrate page at [PeroxigardDisinfectants.com](http://PeroxigardDisinfectants.com)

CONTACT TIMES & DILUTION RATIOS	
DAILY USE (Virucidal*)	BACTERICIDAL FUNGICIDAL PARVOVIRUS
<b>5 Minutes @ 1:64</b> 2 fl. oz./gal	<b>5 Minutes @ 1:16</b> 8 fl. oz./gal
Surface should be visibly wet for the entire contact time.	

## How to Use Peroxigard Concentrate

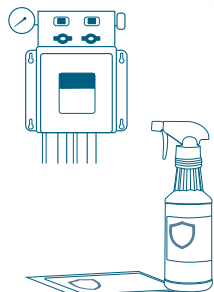

### STEP 1

#### Dilute the concentrate solution

Use a dilution system or approved accessories. Dilute the concentrate at a 1:64 ratio (2 fl. oz. per gallon) for floors, walls or ceilings, or 1:16 ratio (8 fl. oz. per gallon) for above the floor use. Be sure to follow the proper dilution ratios for specific pathogens\*.

### STEP 2

#### Apply a workplace label to any secondary bottles

When diluting into secondary bottles, apply a Peroxigard Concentrate workplace label and record the dilution ratio, lot number and expiry date. The diluted solution remains effective for up to 90 days in a spray bottle.

### STEP 3

#### Pre-clean heavily soiled areas

Remove visible soil and pre-clean the area by working from the least to the most contaminated area.

### STEP 4

#### Apply the diluted solution onto the surface

The entire surface area should be covered and appear visibly wet. Wettability and surface coverage are equally as important to ensure disinfection can be achieved in one application.

### STEP 5


#### Allow surface to remain visibly wet for 5 minutes

In order for disinfectants to be effective they must remain wet on the surface for their entire contact time. Refer to table above for contact times and dilution ratios\*.

### STEP 6

#### Allow to air dry

If desired, you may wipe excess solution using a clean cloth after the required contact time on small surfaces. For larger areas such as flooring, you may conduct a periodic rinse step after the contact time is met.

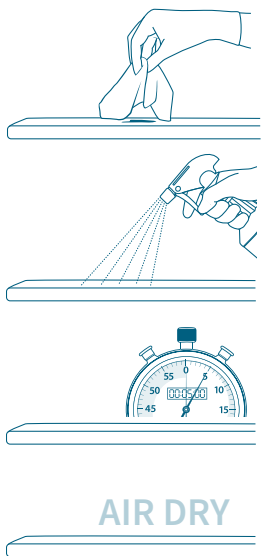

## Success Tips

### Start Fresh.

If you are switching from a previous cleaner and disinfectant to using Peroxigard for the first time, start with fresh accessories and tools. Peroxigard will remove build up from previous disinfectants and may cause a non-toxic reaction and musty odor. The odor will eventually disappear after a few times of using Peroxigard.

### Validate your Dilution.

For the Peroxigard Concentrate solution, there are Precise™ 425 test strips that will provide you confidence that the correct dilution ratio has been achieved. The strips can be used to confirm the calibration of your automated dilution equipment.

### More is NOT Better.

Follow the dilution ratios to conserve product and prevent sticky floors. When diluted properly, Peroxigard Concentrate is a cost-effective solution for your everyday cleaning and disinfection needs.

\*Virucidal at a 1:64 dilution and 5 minute contact time, unless otherwise noted. Refer to product label and reference sheet for full list of claims, contact times and use-directions.



Peroxigard is a member of the Virox family of brands.

DISCLAIMER: You must read and follow the instructions on the label for our product and use the product for the particular applications specified on the product label. To the full extent permitted under applicable law, Virox Technologies Inc. shall not be liable for any damages whatsoever, arising out of or otherwise in connection with any failure to follow those instructions or any use of its products which is not specified on the product's label. The instructions on the product label are not a warranty, express or implied, and Virox Technologies, Inc. expressly disclaims any implied warranties, including any warranty of fitness for a particular purpose. Under applicable law, some of the above waivers or disclaimers of warranty, or exclusions of liability for certain types of damages may not apply to you. In such cases, our liability will be limited to the fullest extent permitted by applicable law. Product shots have been digitally altered and are not accurate as shown. \*See reference sheet for a complete list of efficacy claims, additional features and directions for use.

Accelerated Hydrogen Peroxide® and Design are trademarks of Diversey, Inc.

B-012725-8492-04F

© 2025 Virox Technologies Inc.